

LBBF®

BARRIER CREAM

REF 3821 / 3822 / 3823

CliniMed®



# Barrier Cream

---

## Product Description

LBF Barrier Cream provides an effective barrier to protect intact skin from the harmful effects of bodily fluids, such as stoma output and excessive perspiration.

## Indications for use

LBF Barrier Cream is indicated for the protection of intact skin that is at risk or likely to be compromised by corrosive bodily fluids (including urine and faeces) or adhesives. The product may be used on adults and on children.

## Contraindications

LBF Barrier cream is not intended for direct use on mucosal membranes and broken skin. If in doubt, patch test a small area prior to use.

## Caution

Use only as directed. For external use only. Do not use near eyes, ears, nose or mouth. May not be suitable for use on babies under 30 days old. Use of other barrier products, ointments, creams or lotions may significantly reduce the effectiveness of LBF. Dispose of safely after use.




















## Instructions For Use

1. Skin should be clean and dry prior to application.
2. Apply a uniform coating to the area of concern.
3. Allow LBF Barrier Cream to dry before covering. Once dried, it will not affect any adhesion of further appliances.
4. If LBF is applied to an area with skin folds or other skin-to-skin contact, make sure that skin-contact areas are separated to allow them to dry before returning to normal position.
5. When used under stoma pouches, adhesive tapes or dressings, reapplication is necessary with every change of adhesive appliance as the barrier is removed by the adhesive.
6. When used as a protectant against corrosive bodily fluids (including urine and faeces) and no adhesive products are applied to the skin, reapplication is recommended every 24-48 hours, depending on the frequency of cleaning. In extreme cases (e.g. constant diarrhoeal stooling) with very frequent cleansing, more frequent applications may be necessary (i.e. every 12-24 hours).
7. If desired, the cream can be removed by using most medical adhesive removers.

**Warning:** Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged. Contact CliniMed in the event that the sachet/tube is received damaged before use.

**Ingredients:** Purified Water, Petrolatum, Zinc Oxide, Dimethicone, Cetyl Dimethicone, Cetyl PEG/PPG-10/1 Dimethicone, Isopropyl Palmitate, Propylene Glycol, Bees Wax, Sodium Chloride, Disodium EDTA, Hydrogenated Castor Oil, Methylparaben and Propylparaben

**Storage and Shelf Life:** Store away from direct sunlight at ambient temperature and humidity. For shelf life, refer to the expiry date on the tube or sachet.

	Manufacturer		Date of Manufacture/Country of Manufacture		Keep away from Sunlight
	EC Representative				Do not use if packaging is damaged and consult instructions for use
	Catalogue Number		Caution/Warning		
	Batch Code		Use-by Date		Keep Dry
	Medical Device		Do Not Re-use		Single Patient Multiple Use
	Unique Device Identifier		Not Made with Natural Rubber Latex		Recycling (High-density polyethylene)
	Instructions for Use		Sachet Material Not Yet Recycled		Recycling (Paper and Cardboard)

Product Code	Description	No. Per Box
3821	LBF Barrier Cream Tube, 100g	1 Tube
3822	LBF Barrier Cream Sachets, 2g	1 Box of 20 Sachets
3823	LBF Barrier Cream Tube, 30g	1 Tube